ERGOSURG

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- Feldgenerator (40820030E)

1 Components

- 1 head plate with integrated carriage for mounting the field generator
- 1 headrest pad
- 4 plastic screws M8 for securing the field generator to the carriage
- 1 strain relief for the field generator
- 2 plastic screws M5 for securing the strain relief

2 Intended use

The integrated EM headrest (40820510EU) is intended to position the patient's head during an intervention with an electromagnetic KARL STORZ navigation system (e.g.: 40820001E). It includes a securing mechanism for the field generator (40820030E).

Contraindications

The headrest must not be used if, in the opinion of a responsible physician, the health of the patient is endangered through such use, e.g., due to the general condition of the patient.

Unauthorized conversions, repairs or modifications are not permitted for safety reasons.

WARNING: The headrest must only be subjected to the proportionate load of a patient weighing a maximum of 180 kg.

It is recommended to check the suitability of the products for the intended procedure prior to use.

3 Compatibility

The headrest may only be used together with the following products:

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4 Safety instructions

-  **WARNING:** Always follow the manufacturer's safety instructions in the manuals of the compatible products.
-  **WARNING:** Accessories which have not been secured properly can become loose and result in injuries. Accessories from other manufacturers may only be used with the authorization of KARL STORZ.
-  **WARNING:** Position the patient correctly and observe him or her continuously in order to rule out the risk of compromising vital functions in the event of incorrect positioning.
-  **WARNING:** During HF surgery, suitable additional measures must be taken for insulated patient positioning, as otherwise there is a risk of burns. Insulated patient positioning applies exclusively for HF surgery.
-  **WARNING:** Observe all adjustments closely and prevent collisions. Make sure that no tubes, cables or cloths are trapped.
-  **WARNING:** The product influences the position of the center of gravity on mobile OR tables. For this reason, observe the instruction manual for the mobile OR table during positioning.
-  **NOTE:** The pad is electrically conductive as per DIN EN 60601-2-46.
-  **NOTE:** Check for missing items and evidence of shipping damage. It may be the case that damage only becomes apparent during connection or commissioning. File any complaints immediately with KARL STORZ or the supplier.
-  **WARNING:** The headrest must not be used if components are missing. The headrest must only be assembled and used in its entirety and in the manner described in this instruction manual.
-  **WARNING:** The headrest must only be used by persons who are suitably qualified in patient positioning.

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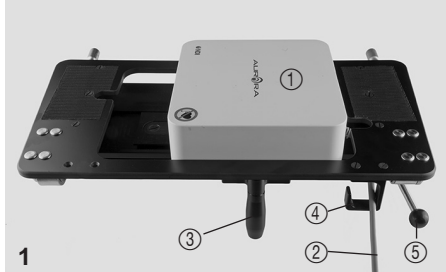
- ! WARNING:** When assembling and adjusting the product, make sure that no one is crushed, caught or injured in any other way and that the accessories do not collide with the surrounding area.
- ! WARNING:** Always follow the manufacturer's safety instructions in the manuals of the compatible products.

5 Assembly

5.1 Attaching the head plate

- Remove the existing head plate from the OR table and secure the integrated headrest on the OR table.
- Insert the spigot on the head plate into the OR table interface. Then tighten the knurled screw on the OR table.
- To position the headrest open the lever (5). You can now move the headrest to the desired position. Then tighten the lever (5) again.

- ! WARNING:** Ensure that the headrest is securely positioned on the OR table.

5.2 Assembling the field generator

- Place the field generator (1) on the carriage in the head plate so that the carriage holes lie over the lateral thread holes of the field generator.
- The cable (2) of the field generator can point to either the right or left depending on which position is more suitable in the OR (Fig. 1).
- Screw the field generator onto the sliding carriage using the 4 plastic screws M8. Tighten the screws hand-tight.

- i NOTE:** Then check that the field generator is positioned securely on the carriage.

5.3 Assembling the field generator strain relief

- Push the field generator on the carriage to the edge of the plate cut-out where the field generator cable does not exit, and guide the cable (2) below the upper plate in the direction of the handle toward the outside (Fig. 2, 3). It may be necessary to unlock the carriage (3) (see chapter 5.4 "Locking the field generator").
- Now guide the cable through the strain relief on the lower end of the cable hook, tighten the cable slightly and screw the strain relief using the 2 plastic screws M5 onto the upper plate (Fig. 4).

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- Check that the cable is securely positioned in the strain relief ④ and ensure that the field generator can be easily moved from the left limit position to the right limit position after securing the cable (Fig. 5).

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5.4 Locking the field generator

Using the screw on the base of the generator holder, the field generator carriage can be fixed in the desired position to prevent it from sliding accidentally.

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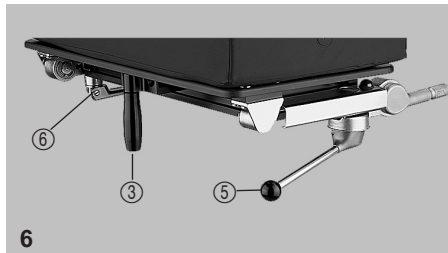
- To access this screw, firstly open the cam lever ⑥ on the side of the headrest (Fig. 6).
- The upper plate, including the field generator, can now be lifted upwards. The headrest opens on the side of the OR table. The ideal position for the field generator in the headrest is in the middle of the sliding rail.
- Once you have locked the field generator, fold down the headrest again and close the cam lever.

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! WARNING: Check that the field generator holder is securely positioned on the headrest.

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5.5 Assembling the headrest pad

Secure the headrest pad in the middle of the head plate (Fig. 7) using the Velcro tape so that the cushion is flush with the plate on every side.

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! WARNING: Check that the pad is securely attached to the head plate.

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To remove the headrest pad, move your flat hand carefully between the pad and the head plate. Pulling on the pad with force damages the bond.

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6 Handling

! WARNING: Never use force to pull the headrest pad from the head plate from one side. Remove the pad by inserting your flat hand between the pad and head plate.

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! CAUTION: Never seal the small valve openings underneath the pad, as they are responsible for compensating the pressure. No liquid can enter into the pad.

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CAUTION: Do not mark the pad using pens or markers as they can leave toxic residue on the surface and the dyes cannot be removed.

WARNING: Replace the pad regularly as foam is subject to a natural ageing process. Over the years, the pad will become softer and this can cause pressure damage to the patient.

CAUTION: Do not attach additional hook and loop tapes to the pad and do not remove the address sticker on the bottom of the pad. This could result in damage to the pad.

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- Check the mechanical parts for signs of damage and wear.
- Check that the adjusting mechanisms function properly. The product must not be used any longer if defects are identified.

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WARNING: Prevent the patient from coming into contact with metal parts and do not position the patient on wet underlays.

WARNING: Use a sterile underlay when positioning the patient.

7 Cleaning and maintenance

7.1 Cleaning

CAUTION: Do not use sharp objects but rather a soft, lint-free cloth or a soft nylon brush.

CAUTION: Do not spray the cleaning agent directly into joints and cracks. Do not use a pressure washer.

CAUTION: Residues from physiological saline solutions attack the surface of the product. Remove such residues using a cloth moistened in clear water and then dry the product with a soft, lint-free cloth.

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WARNING: Make sure to follow the instructions provided by the manufacturer of the cleaning agent and disinfectant as well as the hospital's currently valid hygiene rules.

CAUTION: The headrest must not be reprocessed by machine.

CAUTION: Only use mild cleaning agents and surface disinfectants without aggressive additives.

CAUTION: Only use approved cleaning agents and disinfectants.

Remove all soiling from the product prior to disinfection. Do not immerse the pad in liquid. Dry it using a soft, lint-free cloth.

Perform visual and functional tests each time after cleaning and disinfection (see chapter 8 'Maintenance work and routine tests').

Only use as much cleaning agent as is necessary. Wipe off all excess cleaning agent and disinfectant with a cloth which has been moistened slightly in clear water.

Always wear gloves during cleaning and disinfection.

In order to increase the service life of adhesive and Velcro tape, mechanical cleaning should be avoided.

Only use all-purpose cleaning agents which are slightly alkaline and which contain surfactants and phosphates as the active cleaning components. Do not use solvents such as thinners or acetone to clean the pad.

The following products must **not** be used for cleaning and disinfection:

- Products containing alcohol
- Halides
- Compounds which release halogen
- Products which scratch the surface
- Standard solvents (e.g., ketones)
- Water which contains iron particles
- Cleaning sponges which contain iron
- Disinfectants which contain phenol
- Chlorine
- Hydrogen peroxide
- Products which contain hydrochloric acid
- Products which are strongly basic
- Products which are strongly alkaline

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Integrated EM headrest Model 40820510EU

Remove the pad from the head plate. Carefully move your flat hand between the plate and the pad to release the bonds. Dilute the all-purpose cleaner in clear water as per the instructions given by the cleaning agent manufacturer according to the degree of surface soiling. Slightly moisten a soft cloth with all-purpose cleaning solution and wipe down the product.

 **NOTE:** Particles of dirt can become encapsulated and result in failure to achieve the desired bacteria reduction after disinfection.

Ensure that the product is free from soiling and encapsulated dirt particles.

Slightly moisten a soft cloth in clear water and wipe down the product. Ensure that there is no cleaning agent residue on the product.

Dry the product with a dry, absorbent and lint-free cloth. Wipe-down or spray disinfect the product.

7.2 Disinfection

 **WARNING:** Disinfect the product after each use using wipe-down or spray disinfection. The product must also be cleaned/disinfected before initial use.

 **CAUTION:** Disinfectant residue attacks the surface of the product. Remove such residue using a cloth moistened in clear water and then dry the product using a dry lint-free cloth.

Only use surface disinfectants based on the following active agents for disinfection:

- Aldehydes
- Quaternary compounds

Do not use agents which contain alcohol for high-frequency applications. These can form flammable mixtures which may explode during high-frequency applications.

Do not use disinfectants which contain alcohol to disinfect the pad.

Wipe-down or spray disinfect the product according to the instructions given by the disinfectant manufacturer.

Moisten a soft cloth in clear water and wipe down the product thoroughly. Ensure that there is no disinfectant residue on the product.

Dry the product with a dry, absorbent and lint-free cloth. Perform the visual and functional tests.

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**8 Maintenance work and
routine tests**

WARNING: Perform the following visual and functional tests before each use:

- Check the mechanical parts for signs of damage and wear.
- Check that the adjusting mechanisms function properly. The product must not be used any longer if defects are identified.
- Check that the field generator holder is securely positioned on the headrest.
- Check that the pad is securely attached to the head plate.
- Check the pad for tears and damage.

Furthermore, check to ensure that the product has been reprocessed as per the instructions for cleaning and disinfection before each use.

WARNING: Check the pad for signs of ageing, soft foam and disintegration of the foam.

The product does not require maintenance. However, the safety-relevant functions may be impaired as a result of ageing and wear.

9 Disposal

To dispose of the EM headrests, no special measures are necessary. National laws and regulations must be observed.

10 Directive compliance

This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC.

11 Reusability

If handled with due care and assuming they are neither damaged nor soiled, the headrests can be used again as long as they still function correctly and/or can be assembled. The end of the product's service life is largely determined by wear, reprocessing methods (chemicals) and damage from use. The user is responsible for any other reuse and the use of damaged and/or contaminated headrests. Disregard results in the exclusion of any liability.

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