ERGOSURG

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1 Intended use

The universal EM headrest 40820500EU is intended to secure the field generator to OR tables with a central square mount (or corresponding adaptor solution) using a compatible head plate adaptor. It is used as a basis for positioning the head of the patient before, during and after an intervention in combination with KARL STORZ electromagnetic navigation systems (40820001E).

Contraindications

The headrest must not be used if, in the opinion of a responsible physician, the health of the patient is endangered through such use, e.g., due to the general condition of the patient.

Unauthorized conversions, repairs or modifications are not permitted for safety reasons.

! WARNING: The headrest must only be subjected to the proportionate load of a patient weighing a maximum of 180 kg.

It is recommended to check the suitability of the products for the intended procedure prior to use.

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2 Compatibility

The universal EM headrest may only be used together with the following products:

- Field generator (Art. no. 40820030E)
- The universal EM headrest must only be secured to products which have an appropriate interface or are suitable for attaching an adaptor.

To this end, the safety instructions of the manufacturer should be observed.

3 Safety instructions

-  **WARNING:** Always follow the manufacturer's safety instructions in the manuals of the compatible products.
-  **WARNING:** Accessories which have not been secured properly can become loose and result in injuries. Accessories from other manufacturers may only be used with the authorization of KARL STORZ.
-  **WARNING:** Position the patient correctly and observe him or her continuously in order to rule out the risk of compromising vital functions in the event of incorrect positioning.
-  **WARNING:** Prevent the patient from coming into contact with metal parts and do not position the patient on wet underlays.
-  **WARNING:** The product influences the position of the center of gravity on mobile OR tables. For this reason, observe the instruction manual for the mobile OR table during positioning.
-  **WARNING:** Use a sterile underlay when positioning the patient.
-  **WARNING:** The headrest must not be used if components are missing. The headrest must only be assembled and used in its entirety and in the manner described in this instruction manual.
-  **WARNING:** The headrest must only be used by persons who are suitably qualified in patient positioning.

 **NOTE:** Check for missing items and evidence of shipping damage. It may be the case that damage only becomes apparent during connection or commissioning. File any complaints immediately with KARL STORZ or the supplier.

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4 Assembly

1. Place the field generator on a safe surface and turn it to the emitting side (Fig. 1).
2. Secure an adaptor bracket using two set screws as shown in the figure (Fig. 2).
3. Push the frame into the opening of the bracket (Fig. 3).

! WARNING: Then check that the brackets are securely positioned.

4. Push the second bracket onto the other side of the frame and fix it in place using two set screws (Fig. 4).

! WARNING: Then check that the field generator is positioned securely on the sliding rail.

5. Position the joint in the center of the field generator and secure it in position using the fixing screw (Fig. 5).
6. Pull the pad over the field generator (Fig. 6).
7. Secure the head plate adaptor to the OR table as per the instruction manual of the head plate adaptor manufacturer.
8. Insert the table adaptor into the head plate adaptor and tighten the fixing screw (Fig. 7).
9. Now insert the frame with the field generator and the pad into the designated interface and also tighten the fixing screw (Fig. 8).
10. Tighten all of the screws and ensure that the universal headrest is stable.

5 Adjusting the headrest

! WARNING: Observe all adjustments closely and prevent collisions. Make sure that no tubes, cables or cloths are trapped.

! WARNING: Always make sure that all fixing elements are tightened and the movable parts of the product are correctly fixed in place.

! WARNING: Do not mark the pad using pens or markers as they can leave toxic residue on the surface and the dyes cannot be removed.

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WARNING: Hold the respective individual part firmly before loosening the fixing elements. After each adjustment procedure, make sure that all fixing elements are tightened.

WARNING: When assembling and adjusting the product, make sure that no one is crushed, caught or injured in any other way and that the accessories do not collide with the surrounding area.

1. Hold the field generator securely with one hand. Loosen the fixing screw on the bracket and push the field generator into the desired position.
2. Tighten the fixing screw on the bracket.

WARNING: Then check that the field generator is positioned securely on the sliding rail.

6 Disassembly

1. Remove the universal EM headrest from the OR table as per the instruction manual of the head plate adaptor manufacturer.
2. Hold the field generator securely with one hand. Loosen the 2 plastic screws on each bracket. Remove the field generator.
3. Loosen the fixing screw on the bracket and pull the brackets off the sliding rail.

7 Cleaning and maintenance

7.1 Cleaning

WARNING: Make sure to follow the instructions provided by the manufacturer of the cleaning agent and disinfectant as well as the hospital's currently valid hygiene rules.

WARNING: Disinfect the product after each use using wipe-down or spray disinfection. The product must also be cleaned/disinfected before initial use.

CAUTION: Residues from physiological saline solutions attack the surface of the product. Remove such residues using a cloth moistened in clear water and then dry the product with a soft, low-lint cloth.

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CAUTION: Do not spray the cleaning agent directly into joints and cracks. Do not use a pressure washer.

CAUTION: The headrest must not be reprocessed by machine.

Remove all soiling from the product prior to disinfection and dry it afterwards.

Perform visual and functional tests each time after cleaning and disinfection (see chapter 8 'Maintenance work and routine tests').

Only use as much cleaning agent as is necessary. Wipe off all excess cleaning agent and disinfectant with a cloth which has been moistened slightly in clear water.

Always wear gloves during cleaning and disinfection.

CAUTION: Only use approved cleaning agents and disinfectants.

The following products must **not** be used for cleaning and disinfection:

- Products containing alcohol
- Halides
- Compounds which release halogen
- Products which scratch the surface
- Standard solvents (e.g., ketones)
- Water which contains iron particles
- Cleaning sponges which contain iron
- Disinfectants which contain phenol
- Chlorine
- Hydrogen peroxide
- Products which contain hydrochloric acid
- Products which are strongly basic
- Products which are strongly alkaline

Only use all-purpose cleaning agents which are slightly alkaline and which contain surfactants and phosphates as the active cleaning components. Use a soft, lint-free cloth or a soft nylon brush for cleaning.

Dilute the all-purpose cleaner in clear water as per the instructions given by the cleaning agent manufacturer according to the degree of surface soiling.

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Slightly moisten a soft cloth with all-purpose cleaning solution and wipe down the product.

i NOTE: Particles of dirt can become encapsulated and result in failure to achieve the desired bacteria reduction after disinfection.

Ensure that the product is free from soiling and encapsulated dirt particles.

Slightly moisten a soft cloth in clear water and wipe down the product. Ensure that there is no cleaning agent residue on the product.

Dry the product with a dry, absorbent and lint-free cloth. Wipe-down or spray disinfect the product.

7.2 Disinfection

! **CAUTION:** Disinfectant residue attacks the surface of the product. Remove such residue using a cloth moistened in clear water and then dry the product using a dry low-lint cloth.

Only use surface disinfectants based on the following active agents for disinfection:

- Aldehydes
- Quaternary compounds

Do not use agents which contain alcohol for high-frequency applications. These can form flammable mixtures which may explode during high-frequency applications.

Do not use disinfectants which contain alcohol to disinfect the pad.

Wipe-down or spray disinfect the product according to the instructions given by the disinfectant manufacturer.

Moisten a soft cloth in clear water and wipe down the product thoroughly. Ensure that there is no disinfectant residue on the product.

Dry the product with a dry, absorbent and lint-free cloth. Perform the visual and functional tests.

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8 Maintenance work and routine tests

WARNING: Perform the following visual and functional tests before each use:

- Check the mechanical parts for signs of damage and wear.
- Check that the adjusting mechanisms function properly.

The product must not be used any longer if defects are identified. Furthermore, check to ensure that the product has been reprocessed as per the instructions for cleaning and disinfection before each use.

The product does not require maintenance. However, the safety-relevant functions may be impaired as a result of ageing and wear.

Please check the functions before use. The product must not be used any longer if defects are identified.

9 Disposal

To dispose of the headrests, no special measures are necessary. National laws and regulations must be observed.

10 Directive compliance

This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

11 Reusability

If handled with due care and assuming they are neither damaged nor soiled, the headrests can be used again as long as they still function correctly and/or can be assembled. The end of the product's service life is largely determined by wear, reprocessing methods (chemicals) and damage from use. The user is responsible for any other reuse and the use of damaged and/or contaminated headrests. Disregard results in the exclusion of any liability.

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