ERGOSURG

Navigationsinstrumente

Modelle 40 8000 88EU,
40 8001 10EU, 40 8001 20EU,
40 8001 22EU

Navigation instruments

Models 40 8000 88EU,
40 8001 10EU, 40 8001 20EU,
40 8001 22EU

Instrumentos de navegación

Modelos 40 8000 88EU,
40 8001 10EU, 40 8001 20EU,
40 8001 22EU

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i Hersteller

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Symbol description

! **WARNING:** Failure to observe may result in injury or even death.

! **CAUTION:** Failure to observe may result in damage to or even destruction of the product.

i **NOTE:** Special information on the operation of the instrument.

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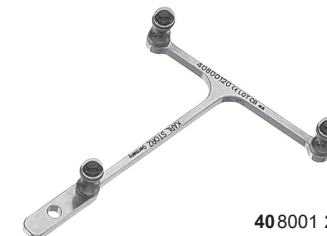
1 Application

It is recommended to check the suitability of the products for the intended procedure prior to use.

! **WARNING:** Remove the instruments from the sterile packaging. Ensure that the individual components are not contaminated (e.g., due to being dropped). If the components are contaminated, they must no longer be used for treatment.

! **WARNING:** The user of the navigation instruments as well as the surgical nurses and other staff must have received authorized training on the safe and effective use of the system from the manufacturer and distributor in order to be allowed to use the KARL STORZ navigation system on patients.

! **WARNING:** The navigation instruments may only be used by persons with an appropriate medical qualification and who are acquainted with the technique of navigated surgery.

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2 Cleaning, disinfection, sterilization, and care

WARNING: Risk of infection: These instruments are not sterile when delivered. The use of non-sterile instruments poses a risk of infection for patients, users and third parties. Inspect instruments for visible contamination. Visible contamination is an indication that reprocessing has not been carried out or has been carried out incorrectly. Reprocess the instruments before initial use and before and after every subsequent use using validated procedures.

WARNING: Risk of infection: Incorrectly reprocessed instruments expose patients, users and third parties to a risk of infection as well as the danger of an instrument malfunction. Observe the 'Cleaning, Disinfection, Care, and Sterilization of KARL STORZ Instruments' instructions and the accompanying documentation.

NOTE: The manual 'Cleaning, Disinfection, Care, and Sterilization of KARL STORZ Instruments' can be downloaded or requested by visiting www.karlstorz.com.

WARNING: A navigation instrument must not be used on patients suffering from Creutzfeldt-Jakob disease. Should the navigation instrument nevertheless be used on such a patient, it must be subsequently disposed of in accordance with the applicable regulations. In this case, the type approval is invalidated immediately.

WARNING: A navigation instrument must be disposed of if it has been used on a patient suffering from a disease for which the mode of transmission is not known.

NOTE: The instrument must not be used if components are missing. It must only be assembled and used in its entirety in the manner described in this instruction manual.

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2.1 General

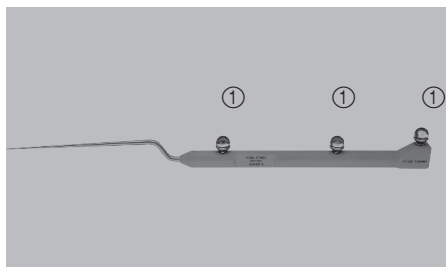
Reprocessing of the instruments must be performed as quickly as possible.

The cleaning, disinfection, and sterilization can be performed either manually or by machine. We recommend performing machine decontamination.

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2.2 Instrument preparation

CAUTION: The glass reflectors ① must not be dismantled under any circumstances!

2.2.1 Disassembling the navigation probe

The navigation probe cannot be dismantled further.

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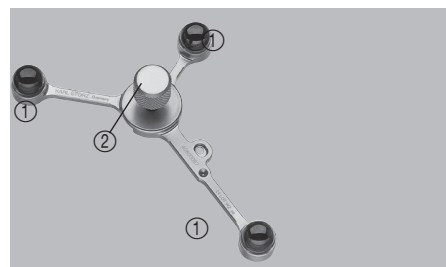
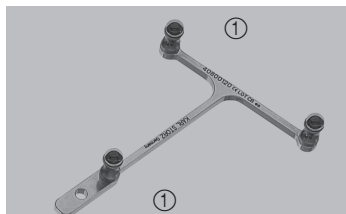
2.2.2 Disassembling the patient tracker

The fastening screw ② of the patient tracker must be completely unscrewed and removed from the thread.

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2.2.3 Disassembling the instrument tracker

The instrument tracker cannot be dismantled further.

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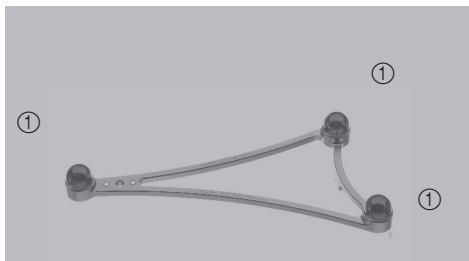
2.2.4 Disassembling the shaver tracker

The shaver tracker cannot be dismantled further.

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3 Reprocessing

CAUTION: Risk of damaging the products: The use of chemicals which have not been approved by KARL STORZ may cause damage to the instruments. Only use the chemicals listed in the section 'Cleaning agents and disinfectants' for reprocessing.

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WARNING: The instruments must be disassembled as described in item 2.2 for reprocessing.

NOTE: When carrying out any work on contaminated instruments, the guidelines of the Employer's Liability Insurance Association and equivalent organizations striving to ensure personal safety must be observed.

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3.1 Preliminary cleaning

NOTE: In order to ensure efficient cleaning, remove heavy soiling, corrosive solutions and pharmaceuticals from the instrument immediately after use in the procedure room, e.g., by wiping down with a surgical pad moistened with distilled water.

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Navigation instruments

Models 40 8000 88EU,
40 8001 10EU, 40 8001 20EU,
40 8001 22EU

In order to dissolve dried organic residues, immerse the instrument in cold water (room temperature) for five minutes.

Clean the surfaces with a sponge or brush under cold running water until all visible contamination has been removed.

CAUTION: The glass reflectors must not be damaged under any circumstances.

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3.2 Manual cleaning/disinfection

Following preliminary cleaning, immerse the instrument (complete coverage with no air bubbles remaining) in a disinfectant solution (see section 4), paying attention to the concentration and exposure times specified by the chemical manufacturer.

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Following the correct exposure time, rinse all parts off thoroughly with microbiologically pure/sterile water. Finally, dry with sterile compressed air.

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3.3 Machine reprocessing

Place the instruments in the holders of the cleaning tray with the glass reflectors facing upwards. In doing so, the glass reflectors must not come into contact with metal parts such as the tray.

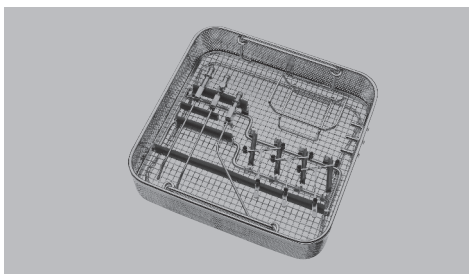
CAUTION: Do not place instruments on the glass reflectors!

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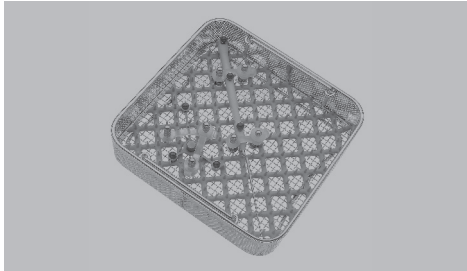
Thermally disinfect the instruments in the washer and disinfectant (≥ 93 °C, device-specific).

CAUTION: The instruments must not be cleaned in an ultrasound bath under any circumstances!

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3.4 Visual and functional check

A visual check for residue must be performed once reprocessing has been completed. Repeat reprocessing steps if necessary.

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3.5 Sterilization

The instruments are always packaged for sterilization. Only cleaned, disinfected, dry and functional medical devices in suitable packaging may be sterilized.

This instrument must be sterilized in its disassembled state at 134 °C – 137 °C with a minimum exposure time of 4 minutes and a maximum of 18 minutes. Steam sterilization using the fractionated prevacuum procedure as per ISO 17665/EN554 is preferable.

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3.6 Check and functional test

Damaged or corroded instruments must not be used further.

The glass reflectors must not display any signs of damage (scratches, etc.) or become cloudy.

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4 Cleaning agents and disinfectants

CAUTION: Do not use solutions predominantly containing alcohol.

CAUTION: Risk of damaging the products: The use of chemicals which have not been approved by KARL STORZ may cause damage to the instruments. Only use chemicals approved by KARL STORZ for reprocessing. A complete list can be found on the internet at www.karlstorz.com/Hygiene.

CAUTION: When preparing and using the solutions, follow the chemical manufacturer's instructions, paying close attention to concentration and exposure times. Prolonged immersion and incorrect concentration may result in damage. Regarding the microbiological effectiveness, please ask the manufacturer.

4.1 For machine reprocessing

For machine reprocessing, enzymatic or mild alkaline cleaning agents with a pH value of up to 11 should be used.

Do not use additives such as cleaning enhancer and rinse aid.

CAUTION: Failure to observe may result in damage to the instruments.

4.2 For manual reprocessing

For manual reprocessing, enzymatic or mild alkaline cleaning agents with a pH value of up to 11 should be used.

Do not use additives such as cleaning enhancer and rinse aid.

CAUTION: Failure to observe may result in damage to the instruments.

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6 Disposal

To dispose of these medical devices, no special measures are necessary. National regulations/laws must be observed.

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7 Directive compliance

In accordance with the Medical Device Directive (MDD): This medical device belongs to Class I

These medical devices bear the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC: 40800088EU and 40800110EU.

These medical devices are CE marked according to the Medical Device Regulation (MDR) 2017/745: 40800120EU and 40800122EU.

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8 Reusability

If handled with due care and assuming they are neither damaged nor soiled, the instruments can be used again as long as they still function correctly and/or can be assembled. The end of the product's service life is largely determined by wear, reprocessing methods (chemicals) and damage from use. The user is responsible for any other reuse and the use of damaged and/or contaminated instruments. Disregard results in the exclusion of any liability.

CAUTION: A damaged or corroded instrument may not be reused and must be replaced.

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